



Links Medical Products Inc.®

**PLEASE OBSERVE THE FOLLOWING BEFORE TAKING ANY TEMPERATURE MEASUREMENT TO ENSURE A STABLE AND RELIABLE RESULT**

**Important note prior to initial use** - the thermometer needs to acclimate to the ambient environment (temperature) in which measurements will occur. It is important to allow the JXB-183 to acclimate to the room temperature for at least 15 to 20 minutes prior to first using the thermometer.

**Subsequent usage** - each time there is a significant change in the ambient temperature due to a change in the environment, allow the JXB-183 to acclimate to this ambient temperature for at least 15 minutes prior to using it.

- Allow 3-5 seconds interval between each measurement
- Push hair away from the forehead
- Wipe away any perspiration from the forehead
- Avoid any drafts (i.e. heater or air conditioning)

Refer to the user manual for further instructions.

A handwritten signature in black ink that reads 'Patrick Buckley'.

Patrick Buckley  
President



***your link to innovative caring products***